

Melladerm® Plus, Antibacterial Wound Gel (use 1-2 days)

For the treatment of (contaminated) chronic, oncologic and/or acute wounds. Melladerm® Plus is designed specifically with the patients with necrotic or slough wounds. Melladerm® Plus offers a more gentle approach than pure honey that is known to cause pain when applied to the wound. It contains 45% honey, which is more than Melladerm® ointment. The honey based wound gel will not cause pain and utilises the antibacterial strengths of honey. The honey source of Melladerm® Plus is Bulgaria. This is specially selected honey from a multiflower mountain region that has a naturally high glucose oxidase and Phenolic content.

During its processing into a wound gel this honey is not heated or irradiated because this is known to destroy the honey healing properties. (please see expert document for in-depth information). The Bulgarian honey has a very high peroxide level (strong antiseptic) and is sterilized with a by SanoMed Manufacturing bv patented method (ozonation). The same strict selection criteria are used as for the fynbos honey in the Melladerm®.

Melladerm® Plus can be used to fill the wound cavity and then covered with SanoSkin® Net or Foam. The gel is easy to apply and will not adhere to the wound. Melladerm® Plus has more debriding capabilities than Melladerm® and has unique healing properties. Melladerm® Plus comes also in plastic tubes of 20 or 50 gram.

Product Code	Description	Contents	Contents over box
MDP20	Melladerm Plus 20 gram	1 tube à 20 g	12 x 1 tube
MDP50	Melladerm Plus 50 gram	1 tube à 50 g	12 x 1 tube



Case: A 70 year old man had an infected wound above his eye lid. The wound was filled daily with SanoSkin® Melladerm® Plus. The Melladerm® Plus was covered with SanoSkin® Foam and secured with light tape. It took 20 days to heal the wound completely.





Case: The Wound (mixed venous/arterial ulcer) was treated with Melladerm® Plus and SanoSkin® Foam for 50 days. The patient did not report pain or discomfort during the treatment. The wound existed for over 3 years and the patient was in bad condition, still the wound healed. Melladerm® Plus was applied daily.



Case: A 81 year old lady had a venous ulcer for over 3 years and was treated mainly with Hydrocolloids and Betadine ointment. The wound never healed. The doctor started a treatment with Melladerm® Plus and the wound quickly cleaned up and started to granulate. It took 8 weeks to reach the result shown in the picture. Compression therapy was also part of the treatment and a SanoSkin® Foam was used as secondary dressing.