

## SanoSkin® Cleanser

SanoSkin®-Cleanser is a solution to clean wounds. "Practitioners continue to use antiseptics in wounds because of tradition – This tradition must stop !!!" (Dow G. Infection in chronic wounds. In: Krasner DL, Rodeheaver GT, Sibbald RG, eds Chronic Wound Care: A clinical Source Book for Healthcare Professionals. 3th ed. Wayne, Pa: HMP Communications, 2001: 343-356.)

Modern wound treatment recommend a cleansing solution based on physiologic saline mixed with mild detergents (non cytotoxic levels) that effectively remove proteins and fat. The presence of glycerine in the cleaning solution makes it a gentle soft solution and the chlorhexidine preserves the solution for several years from microbiological contamination. The SanoSkin® Cleanser formula is being used for over 7 years with great satisfaction. Its easy nozzle spray system makes it possible to adjust the cleaning power (output of the cleaning stream) so even hard to clean wounds can be cleaned. The bottle contains 250 cc. To remove slough, leave a swap with SanoSkin cleanser solution for 5 minutes on the wound.

Product Code	Description	Contents	Contents over box
SC250	SanoSkin-Cleanser	bottle 250 cc	6 x 1 bottle





Oplossing voor wondspoeling  
Solution de nettoyage de plaies  
Wound cleanser

reinigend	nettoyant	Cleansing
efficiënt	efficace	Efficient
veilig	sûr	Safe
gemakkelijk	pratique	Easy to use

Uitwendig gebruik - Usage Externe  
External usage



**SanoSkin Cleanser**  
**Indikaties    Indications**

Verwijdert debris en micro-organismen, reinigt en spoelt  
Enlève débris et micro-organismes, rince et nettoie  
Removes debris and micro-organismes, rinses and cleans

<b>Gebruiksaanwijzing</b>	<b>Mode d'emploi</b>
Zet spraykop in "Spray" (irrigatie) of "stream" stand (reïning)	Mettez le bec en position "spray" (irrigation) ou "stream" (nettoyage)
Zet na gebruik steeds weer in "off" stand	Mettez le bec en position "off" après chaque utilisation

**Mode of action**  
Turn nozzle head to spray (to irrigate) or stream position (to clean)  
Turn nozzle to off position when finished

<b>Waarschuwing</b>	<b>Précautions</b>	<b>Warning</b>
Niet als antisepticum gebruiken Do not use as an antiseptic		Ne pas utiliser comme antiseptique

**INGREDIENTS**  
Aqua, Polysorbate 20, Glycerol, Sorbitol, Chloorhexidine digluconaat, Sodium hydroxide.

250 ml e

Manufacturer  
SanoMed Manufacturing B.V.  
Smoutweg 4-008  
4524 EK Sluis  
The Netherlands



With the wound cleanser it is possible to remove the necrotic tissue and consequently the microbiological contamination. A physical removal is usually removing 95% of the bacteria and slough.