

## SanoSkin®-NET Product Sheet

**SanoSkin®-NET** is developed to be used for medical applications as a primary wound dressing. SanoSkin®-NET consists of a polyester woven net, that is coated with a transparent hydrogel that is able to absorb excess exudate up to 15 times its own weight. The dry gel sticks to the dry skin, but when moisturized (wound fluid), the gel does not stick to wet surfaces (wound). It will create a moist wound healing environment and acts bacteriostatic (stops growth of bacteria in the wound). SanoSkin®-NET provides a gentle and painless way of healing superficial wounds. The extra gel layer absorbed the first wave of fluid and when more wound fluid is produced this will pass through the perforations and keep the wound in an ideal moist environment. Meanwhile secondary dressings are not able to stick into the wound (interface function). The Net can be used in conjunction with other ointments or gels such as **Melladerm®**, **Melladerm® PLUS** and **SanoSkin®-OXY**. The sizes are 8,5 x 12 cm, 12 x 15, 10 x 18, 8 x 10, 5 X 7,5 cm and 20 x 20 cm.



Product Code	Description	Contents	Contents over box
SN85120	SanoSkin-NET 8,5 x 12 cm	5 x 1 piece	60 X 5 x 1 piece
SN200200	SanoSkin-NET 20 x 20 cm	5 x 1 piece	30 x 5 x 1 piece
SN120150	SanoSkin-NET 12 x 15 cm	5 x 1 piece	30 x 5 x 1 piece
SN100180	SanoSkin-NET 10 x 18 cm	5 x 1 piece	30 x 5 x 1 piece
SN5075	SanoSkin-NET 5 x 75 cm	5 x 1 piece	60 x 5 x 1 piece



**Case:** A 77 year old lady developed a heel blister due to friction and after all the death skin was removed the wound was treated with SanoSkin® NET and some SanoSkin® Melladerm®. It took 38 days to full epithelialisation. Before removal of the SanoSkin® NET the wound was soaked with some saline solution.



**Cases:** Different examples were SanoSkin® NET can be used. Big burns or gravel rash can be cleaned with SanoSkin® Cleanser and then a SanoSkin® Net can be applied and Left for three days. A gauze dressing can be used on top to take away the excess wound fluid, but since the Net is able t absorb a lot of fluid itself, even the top dressing can stay in place long enough. A simple bandage can secure the whole construction. Removal of the SanoSkin® Net must be done after wetting it with Saline solution or one can use the SanoSkin® Cleanser for that too.

