


## SanoSkin® -FIRST Product Sheet

The **SanoSkin® First** is developed for a quick and efficient treatment of small superficial wounds. SanoSkin®-First is a transparent Hydrogel sheet that is able to absorb excess exudate up to 15 times its own weight. It will create a moist wound healing environment and acts bacteriostatic (stops growth of bacteria in the wound). It is covered by a poly urethane film + overlap border and therefore acts as an antibacterial barrier. The special design allows the patient to shower and even swim. The dressing can stay on the wound for several days and allows visual inspection without removing the dressing.

The moisture vapor transmission rate (MVTR) of the hydrogel is 1020 g/m<sup>2</sup>/24 hours. This dressing is ideal for small burns, abrasions and cuts. The size is 74 x 45 mm (with PU overlap), oval shaped, and is packed in a box of 5.

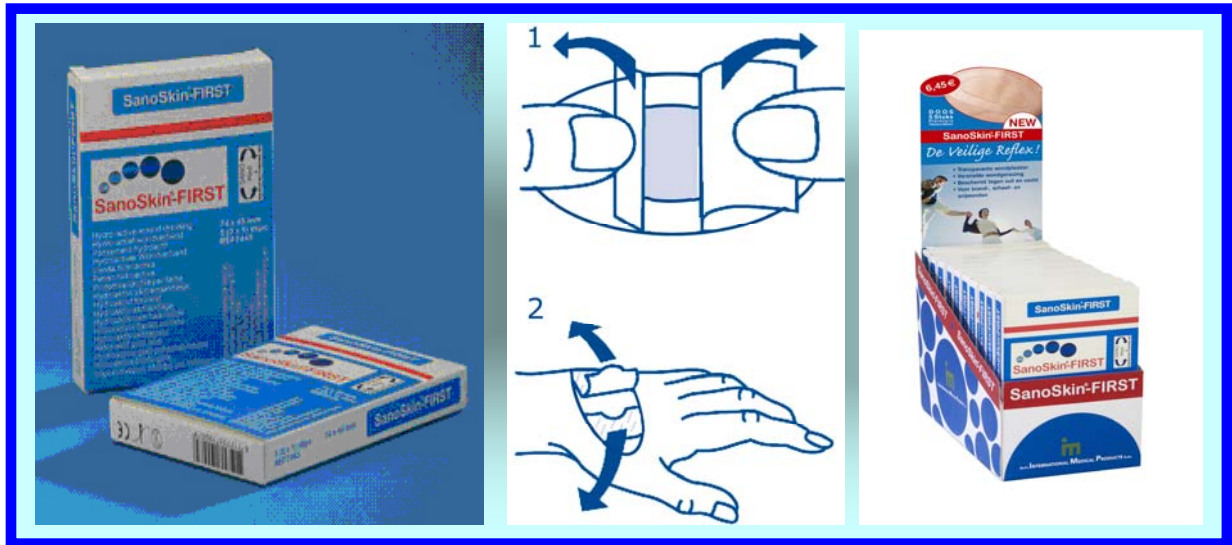

Product Code	Description	Contents	Contents over box
SF7445	SanoSkin-First 74 X 45 mm	5 x 1 piece	60 X 5 x 1 piece



**Case:** Small superficial burn wounds can also be dressed with SanoSkin® First. This hot water burn located at the elbow is after cooling and drying treated with SanoSkin® First and left for 5 days. Even this difficult to dress localisations can be treated with the island dressing.



**Case:** Superficial abrasion wound in an in line skating champion (photo's taken by RED CROSS Belgium, during European championship August 2007)

Wound is cleaned and dressed with a SanoSkin®- First and left for 2 days.

The usual bleeding for the first hour is stopped and the dressing is able to absorb the excess exudate and stays in place so the athlete could continue the championship and shower afterwards. The dressing can easily be removed and after 4 days the wound is almost healed. Only two SanoSkin®-First dressings were used to treat this wound. SanoSkin® First became the first choice dressing for the European in line skating championship organisation. The Red Cross also started to use SanoSkin® First as a standard First Aid dressing.